

# Adjustable Bath Step

**UBAS0003**



---

**USER MANUAL**

---



**WARNING:** Please read this Manual carefully before using the adjustable bath step as it contains important information regarding the safe use of the product.

# Contents

1.	Description of product .....	4
2.	Instructions of use .....	4
3.	Cleaning Instruction .....	6
4.	Technical Specifications .....	6
5.	Warranty .....	6
6.	Contact Us .....	7

## **1. Description of product**

This bath step has been designed as a versatile modular adjustable platform bath step. It features four 1" high sections that can be joined together to build up a secure & stable platform.

The step is supplied with four 1" (25mm) sections giving a basic step of 4" (100mm) but can be added to with further sections up to a maximum height of 8" (200mm), subject to your own risk assessment. Each section has four strong locator pins that fit firmly into the section below.

The bottom section has four rubber feet to prevent any slippage of the step-in use. The top area of each section is textured to provide a slip reducing surface, without making it uncomfortable on the feet.

## **2. Instructions for use**

To prevent any overbalancing problems when either stepping into or out of the bath, please ensure that the step height is equal to the height of the lowest point inside the bath. This will allow both feet to be at the same level whether stepping in or out of the bath.

**WARNING:**

- Avoid sudden or undue movement whilst using this equipment.
- Ensure the step is dry, free from liquid, soap, oil film or any slippery substance.
- Before using the bath step, ensure all sections are firmly pushed together.

**CAUTION:**

- Position step against the side of the bath
- Stand centrally on the step with both feet
- Step directly in to the bath

**NOTE:**

- Test for stability before use
- The product is designed for use in a static mode only

### 3. Cleaning Instruction

Use a non-abrasive mild detergent along with warm water & a soft cloth to clean the step. Rinse with clean warm water & a soft dry cloth.



**CAUTION:**

- Do not use any form of abrasive cleaner, cloth or sponge as this could cause damage to the product.

### 4. Technical specification

<b>Maximum user weight:</b>	500Kg/ 78.5 Stone
<b>Length:</b>	450mm
<b>Width:</b>	35mm
<b>Height (per section):</b>	25mm
<b>Kit height:</b>	100mm

**NOTE:**

- 4 sections per kit
- \*200mmH (max permitted height i.e 2 kits)

### 5. Warranty

This product is guaranteed to be free of defects in materials or workmanship for one year from date of purchase.

This product has been built to high standards of workmanship and has been carefully inspected prior to shipment. This warranty is an expression of the confidence we have in the workmanship and materials used in our products. In the unlikely event of there being a defect found with the product, please contact your supplier and they will either replace the product or supply parts for the product to be repaired at their discretion.

This warranty does not cover failure of the device caused by misuse, negligence or fair wear and tear of the product.

## 6. Contact Us

Ultimate Healthcare customer service is available to answer questions from 8.30am to 5.00pm, Monday to Friday for business and upgrades. Our sales team is available for pricing and order processing and status queries. Our Technical Support team is available to answer your queries related to troubleshooting, please contact the following:

### **Ultimate Healthcare Limited**

Customer Service, Support, Sales Enquiries and Technical Support  
B.B.I.C, Unit 1  
Innovation Way  
Wilthorpe,  
Barnsley,  
S75 1JL

Tel: 0333 321 8996

Email: [sales@ultimatehealthcare.co.uk](mailto:sales@ultimatehealthcare.co.uk)

**Please Note:** Information in this user manual is subject to change without notice and does not represent commitment on the part of Ultimate Healthcare Limited. No part of this manual may be reproduced or transmitted in any form or by any means, electronic or mechanical, including photocopying and recording for any purpose without the written permission of Ultimate Healthcare

Ultimate Healthcare Limited.  
Calmore Industrial Estate  
Totton, Southampton  
SO40 3WW  
United Kingdom



Doc no.  
UBAS0003.IFU-2

Issue 2.0  
February 2022  
© Ultimate Healthcare Limited 2022